

Microphone EM 103

Electret Microphone
for hearing perception training (HPT)



Replacing the battery:

Keep the microphone on the head and unscrew the sleeve, which is then pushed over the cable. Insert the new battery in the correct direction and screw the sleeve back. Use only high quality batteries.

Cautions:

- Switch the microphone off when not in use.
- The cable is about 6 m long. Allow unused cable length combined.
- The foam article should always be used to reduce air noise when talking. These can be ordered from us.
- The microphone should be 5 to 15 cm from the mouth. Do not try to compensate by turning up the volume control when the microphone is further away, as noise to be strengthened.
- Set the microphone with a table stand (MS 1) may on the side, if a book is before the user or use a floor stand (MS 6), which is placed on the side of the table.

AUDIVA 2010

many thanks for google translation service - if you know some better readings, let us know!